


INNOSMILE Workflow

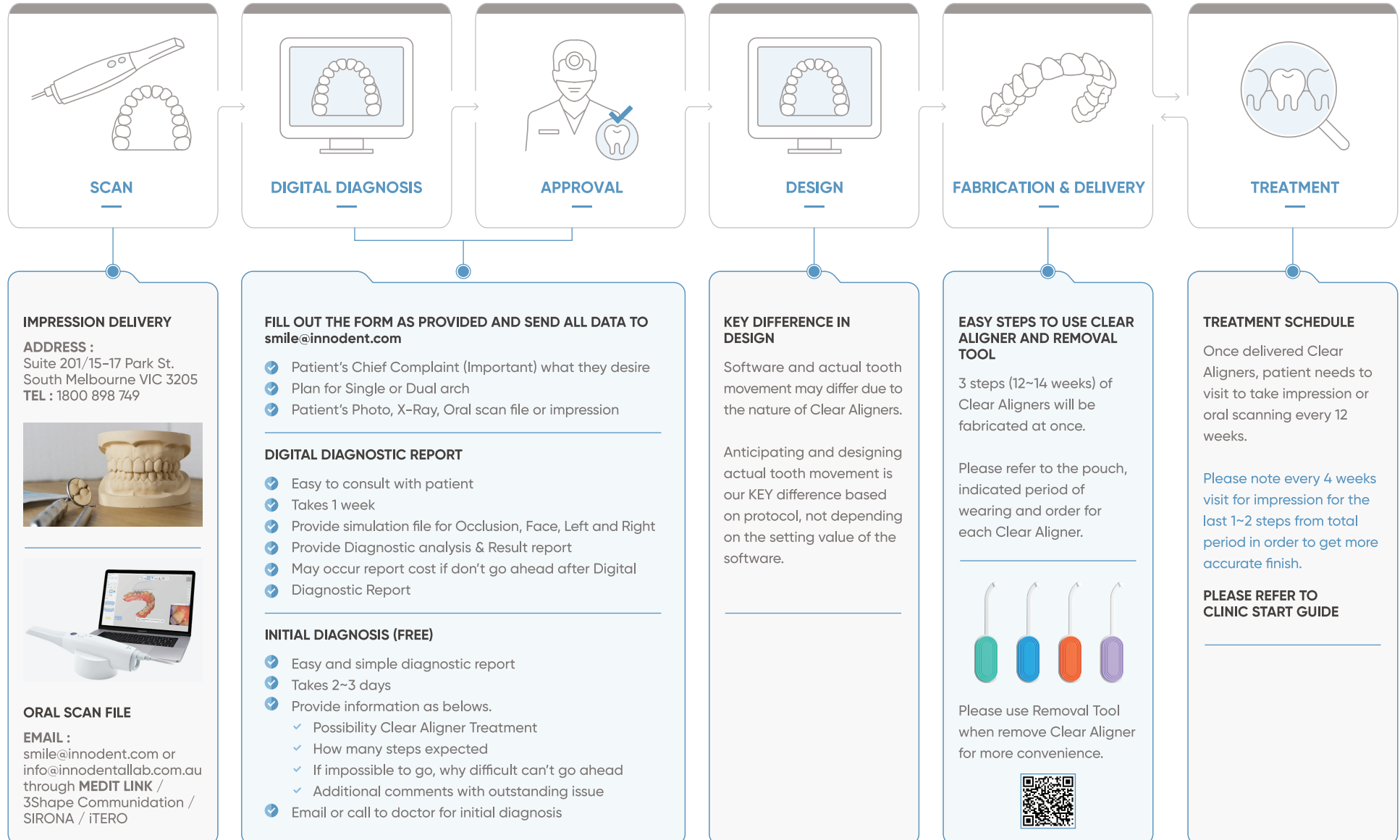

01



BEFORE TREATMENT

1. CHECK CLEAR ALIGNERS AND PACKAGE

- ✓ Patient Name, Number of Clear Aligner, Starter Kit (Case, Remover Tool, Instruction) on the package
- ✓ Clear Aligner Report
- ✓ Attachment template (Refer to report)

2. ARRANGE APPOINTMENT WITH PATIENT

- ✓ Allow more treatment time if IPR (Interproximal Reduction) and attachments on Clear Aligner need to be proceeded. Please inform treatment time to patient in advance

3. CLEAR ALIGNER REPORT

- ✓ Checking Clear Aligner Report for IPR (Interproximal Reduction), attachments on surface, tooth number with location before treatment makes to be convenient for consultation with patient
- ✓ Clear Aligner Report makes you easier to consult and treatment when patient visits

4. CONSULTATION

- ✓ Explain a short version of instruction in Starter Kit
- ✓ Explain IPR (Interproximal Reduction) and attachments
- ✓ Discuss regular visit schedule for next steps, relating number of Clear Aligners for each step

02



TREATMENT

1. ATTACHMENTS

- ✓ Check Clear Aligner Report, tooth number & location
- ✓ Prepare resin to attach, then proceed following steps
 1. Place soft Clear Aligner on patient's designated arch and check fitting properly
 2. Check template suitability before attaching
 3. Oral prophylaxis + Etching + Bonding on Labial surface
 4. Inject Resin inside of template
 5. Placing template
 6. Use tweezer to place correctly
 7. Curing
 8. Remove residue resin on the surface by using round bur
 9. Fitting Clear Aligner

[PATIENT – CAUTION]

- ✓ It may occur worn out or falling a part of resin while removing Clear Aligner, if then can be redo attachments

2. IPR (INTERPROXIMAL REDUCTION) PROCEDURE

- ✓ Refer to memo to check number of tooth and location
- ✓ Prepare IPR System Kit, Ortho Strip Handpiece, Space Gauge (0.1mm unit)
- ✓ When proceed contact point open by high-speed handpiece, recommend direction from lingual side to labial side due to prevent aesthetic problem
- ✓ For excessive overlapping teeth – recommend proceeding with division rather than IPR at once
- ✓ Recommend use fluoride application after IPR (Interproximal Reduction)

03



AFTER TREATMENT

1. GUIDE PRECAUTION

- ✓ Patient's cooperation is very important for whole treatment
- ✓ Guide to use instruction of user with starter kit
- ✓ Guide precautions

2. GUIDE HOW TO WEAR AND REMOVE, MAINTENANCE

- ✓ Simulate how to wear and remove Clear Aligner
- ✓ Recommend to use Chewie with Removal tool to seat properly after wearing Clear Aligner
- ✓ Keep used Clear Aligner which is finished steps in case of reuse if lost or less movement
- ✓ Recommend to wear Clear Aligner over 20 hours except eating and oral care

3. CHECK FOR NEXT APPOINTMENT

1. Checking comfortability
2. Checking fit of the last Clear Aligner before appointment
3. Checking resistance when attaching and detaching
4. Checking attachments: location, maintenance & worn-out
5. Checking IPR (Interproximal Reduction) area whether less movement due to lack of space
6. If needed, take a photo for space, crowding, occlusion, etc.
7. Checking any fracture or damage on Clear Aligner
8. Periodontal and oral care status
9. Checking next Clear Aligner process and scheduling with patient
10. Leave a note if needed impression or scan, photos for next step

4. REFINEMENT

- ✓ Compensation for insufficient movement after wearing all Clear Aligners of the first diagnosis quantity
- ✓ Free additional refinement under package deal
- ✓ Should order during whole treatment period (Ex. If 6 months treatment schedule, should order within 6 months)